THUASNE

SPORT

fr	Chevillère ouverte Thuasne Sport.....	3
en	Thuasne Sport open ankle support	4
de	Knöchelbandage mit offener Ferse Thuasne Sport.....	5
nl	Thuasne Sport open enkelbrace	7
it	Cavigliera aperta Thuasne Sport.....	8
es	Tobillera abierta Thuasne Sport	9
pt	Tornozeleira aberta Thuasne Sport.....	11
da	Åbent ankelbind Thuasne Sport	12
fi	Avoim Thuasne Sport -nilkkatuki	13
sv	Thuasne Sport öppen ankelortos	15
el	Επιστραγαλίδα με ανοικτή πτέρνα Thuasne Sport....	16
cs	Kotníková bandáž s otevřenou patou Thuasne Sport	17
pl	Otwarta orteza skokowo-stopowa Thuasne Sport.....	19
lv	Thuasne Sport atvērtais potītes pārsējs	20
lt	Atviras kulkšnies įtvaras „Thuasne Sport“	21
et	Thuasne Sport avatud hüppeliigese tugi	23
sl	Odperta opornica za gleženj Thuasne Sport	24
sk	Otvorená ortéza členku Thuasne Sport	25
hu	Thuasne Sport nyitott bokarögzítő	26
bg	Отворена ортеза за глезен Thuasne Sport	28
ro	Glezniară deschisă Thuasne Sport.....	29
ru	Открытый голеностопный ортез Thuasne Sport	31
hr	Otvoreni steznik za gležanj Thuasne Sport	32
zh	Thuasne Sport开放式护踝	33
ar	Thuasne Sport دعامة كاحل مفتوحة	36

	S 19 - 20	M 21 - 22	L 23 - 24	XL 25 - 27
---	---------------------	---------------------	---------------------	----------------------

fr	Tour de cheville	_____
en	Ankle circumference	_____
de	Knöchelumfang	_____
nl	Omvang van de enkel	_____
it	Giro caviglia	_____
es	Circunferencia del tobillo	_____
pt	Perímetro do tornozelo	_____
da	Anklens omkreds	_____
fi	Nilkanympärys	_____
sv	Runt fotleden	_____
el	Περιμετρος αστραγάλου	_____
cs	Obvod kotníku	_____
pl	Obwód kostki	_____
lv	Potītes apkārtmērs	_____
lt	Kulkšnies apimtis	_____
et	Pahkluu ümbermõõt	_____
sl	Obseg gležnja	_____
sk	Obvod členka	_____
hu	Boka kerülete	_____
bg	Обиколка на глезена	_____
ro	Circumferința gleznei	_____
ru	Окружность голени	_____
hr	Opseg gležnja	_____
zh	脚踝尺寸	_____
ar	محيط الكاحل	_____

fr

CHEVILLÈRE OUVERTE THUASNE SPORT

Description/Destination

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THUASNE SPORT OPEN ANKLE SUPPORT

Description/purpose

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

CR neoprene - Polyamide - Elastane.

Properties/Mode of action

Limitation of the heat loss by the synthetic rubber (Chloroprene).

Opened at the heel part.

Indications

Joint instability/laxity.

Proprioceptive support for rehabilitation.

Mild ankle sprain (acute phase).

Sport activities resumption.

Contraindications

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Precautions

Read the instruction leaflet carefully.

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Consult a healthcare professional:

- In case of doubt before use,

- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,

- In case of changes in the device's performance.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Pull the ankle brace over the ankle joint, like a sock.

Make sure the opening fits over the heel.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Hand wash. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2033401 (2022-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186