THUASNE

SPORT

fr	Genouillère ouverte Thuasne Sport	3
en	Thuasne Sport open knee brace	4
de	Offenene Thuasne Sport-Kniebandage	5
nl	Thuasne Sport open kniebrace	7
it	Ginocchiera aperta Thuasne Sport	8
es	Rodillera abierta Thuasne Sport	9
pt	Joelheira aberta Thuasne Sport	11
da	Åbent knæbind Thuasne Sport	12
fi	Avoin Thuasne Sport -polvituki	13
sv	Öppet knästöd Thuasne Sport	14
el	Επιγονατίδα ανοικτή Thuasne Sport	16
cs	Otevřená kolenní ortéza Thuasne Sport	17
pl	Orteza kolanowa otwarta Thuasne Sport	18
lv	Atvērtā ceļa ortoze Thuasne Sport	20
lt	Atviras kelio įtvaras „Thuasne Sport“	21
et	Thuasne Sport avatud põlvekaitse	22
sl	Odperta opornica za koleno Thuasne Sport	23
sk	Otvorená kolenná ortéza Thuasne Sport	25
hu	Thuasne Sport nyitott térdmerevítő	26
bg	Отворена наковенка Thuasne Sport	27
ro	Genunchieră deschisă Thuasne Sport	29
ru	Открытый наколенник Thuasne Sport	30
hr	Otvoreni steznik za koljeno Thuasne Sport	31
zh	Thuasne Sport 开放式护膝	33
ar	Thuasne Sport داعم الركبة المفتوح	35

 cm	S 31 - 33	M 34 - 36	L 37 - 39	XL 40 - 42	XXL 43 - 45
--	---------------------	---------------------	---------------------	----------------------	-----------------------

fr	Circonférence de genou en légère flexion
en	Knee circumference in slight flexion
de	Knieumfang bei leichter Beugung
nl	Omvang van de licht geplooidie knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Ceļgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenкто kelio apimtis
et	Kergelt pinguldatult põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
ru	Обхват колена в состоянии небольшого сгибания
hr	Obuhvaća koljeno u blagoj fleksiji
zh	轻度弯曲的膝关节周长
ar	محيط الركبة في حالة الثني الخفيف

fr

GENOUILÈRE OUVERTE THUASNE SPORT

Description/Destination

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THUASNE SPORT OPEN KNEE BRACE

Description/purpose

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 5 sizes.

The device is composed of a synthetic rubber (chloroprene) tissue with an opening on the kneecap.

Composition

synthetic rubber (chloroprene) - polyamide - elastane

Properties/Mode of action

Limitation of the heat loss by the synthetic rubber (chloroprene).

Indications

Knee pain and/or swelling management.

Joint instability/laxity.

Sport activities resumption.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Pull the knee support.

Centre the front opening over the kneecap.

Thuasne Logo must be on the upper part, centered and focused on the patella.

Care

Hand washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

Keep this instruction leaflet

de

OFFENENE THUASNE SPORT-KNIEBANDAGE

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©Thuasne - 2035301 (2022-08)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186