**THUASNE**

# Hip-med

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GB/USA: Flexion abduction splint

F: Harnais d'abduction

I: Divaricatore d'anca

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DK: Bøj-sprede-ortesen

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**Flexion abduction splint**

This product requires qualified and individual adaptation to the patient by specialist staff.

**Purpose**

The Hip-med® is an orthosis designed to correct the position of the femoral head in the hip socket by spreading the thighs. The orthosis is only to be used to treat the hips and is intended exclusively for young children.

**Description**

The Hip-med® orthosis consists of an adjustable strap system, an infinitely variable abduction splint and the appropriate thigh shells. The strap system secures the flexion of the hip joints and the abduction splint prevents the adduction and secures the specified abduction. Hip-med® stabilises the legs in a 90° to 100° physiologically natural flexion and in an abduction of approx. 50°. Thus the splint prevents every form of stretching and abduction movements in the hip joints which may cause dislocation. It prevents the thighs from sinking into the Lorenz-position. The orthosis deliberately allows evasive movements into further flexion and rotation. The flexion-abduction splints can be worn directly against the body or over clothing.

**Indications**

Hip dysplasia, endangered hip dislocation, repositioned hip.

**Contraindications**

Non repositioned hips.

**Side effects**

None reported when applied and worn as intended.

**Fitting instructions**

(1) Arrange the splint with the blue shoulder strap system facing you. Open the white closure in the chest area. Open the two front easy-click buckles at the thigh shells.

(2) Spread the splint out completely in front of you.

(3) Lay the infant on the strap system and put the belts over the shoulders. Position the thighs of the infant in the thigh shells.

(4) Close the front buckles at the thigh shells and then the white chest buckle.

**Please make sure that the buckles at the thigh shells are locked in place properly!**

(5a) The white chest buckle should be positioned as low as possible.

(5b) Please position the blue shoulder pad as high as possible in the neck area.

(6) The belts are fitted with twin-buttons of different diameters at the front and back part. The bigger front part shall stay in the belt.

(7) To adjust the right length of the belts loosen the smaller back button, find the required length and close the button.

Special feature: the knurled head screw (R) permits a continuous adjustment of the abduction without any tool.

**Precautions**

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its

original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

**Care instructions**

The pads of the shoulder strap system and of the thigh shell can be removed and washed at 30°C. An additional shoulder pad is included. The shoulder straps, the belts and the abduction unit can be cleaned with a moist cloth.

**Material composition**

<b>Neck Part</b>	
U-shaped neck cushion	Terry cotton
Blue shoulder pad	EVA foam
White 10mm front buckle	Polyamide
All Velcro fastenings	Polyamide
Edging material, 16mm, elasticated	85% polyamide, 15% elastane
<b>Straps</b>	
Double buttons	Polyamide 6.6
Straps	80% polyamide, 20% polyurethane
<b>Positioning shells</b>	
Thigh shells	Polyethylene
Spreading unit	Stainless steel
Thread anchors	Brass
Fasteners	Polyamide
Shell padding	Terry cotton
Edging material, 16mm, elasticated	85% polyamide, 15% elastane

Sizes	Age	Abduction width	Thigh circumference
<b>XXS</b>	Premature infants	11,5 - 14 cm 4 1/2 - 5 1/2"	12 - 16 cm 4 3/4 - 6 1/4"
<b>XS</b>	New-born	13 - 16 cm 5 1/4 - 6 1/4"	12 - 16 cm 4 3/4 - 6 1/4"
<b>S</b>	Few weeks old	13 - 18 cm 5 1/4 - 7"	15 - 18 cm 6 - 7"
<b>M</b>	Several weeks	14,5 - 20,5 cm 5 3/4 - 8"	17 - 22 cm 6 3/4 - 8 3/4"
<b>L</b>	Several months	20 - 27 cm 7 3/4 - 10 3/4"	20 - 25 cm 7 3/4 - 10"

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
## THUASNE DEUTSCHLAND GmbH